Please observe the following instructions to assure continued trouble-free and safe product operation:

CONNEX L
4-hole connection with contact pins.
With anti-retraction valve.
With integrated light bulb via
2-pin electrical connection.
Bulb operating voltage max.3.3 Vd.C.

CONNEX GR
4-hole connection with glass rod.
With anti-retraction valve.

IMPORTANT REMARKS

The CONNEX™ Coupling Devices are:

- only to be used by a qualified dental practitioner, in connection with the provision of dental treatment to dental patients. Any other use may prove dangerous and is strictly forbidden.
- a medical product according to the applicable medical device regulations.

The CONNEX™ Coupling Devices may only be used by a competent person for the described application, and under consideration of:

- valid regulations on work safety,
- valid accident prevention measures,
- these operating instructions.

It is the duty of the user:

- to use only faultless tools,
- to ensure that it is used for the correct purpose,
- to protect patients and third parties from all dangers.

OPERATIONAL INSTRUCTIONS

Connection to Hose:

1. A supply of dry & clean compressed air according to EN ISO 7494 – 2 must be provided.
2. Screw the CONNEX™ Coupling Device onto the dental unit's tubing and finger tighten onto the threaded sleeve of the handpiece tube. Then tighten fully with key supplied.

3. Lightly spray the O-rings with STATCARE™ Spray. The coupling remains permanently screwed to the hose.

Changing the STATIS™ Handpieces:

1. Press and rotate the STATIS™ handpiece onto the CONNEX™ Coupling Device and push until it clicks audibly into place.
2. Check to be sure the handpiece is properly connected.

TROUBLESHOOTING


Replacing the Bulb (CONNEX L):

1. Unscrew the cap surrounding the bulb by turning it counter-clockwise.
2. Pull the bulb out of the receptacle.
3. Carefully insert new bulb – Note the two contacts in the coupling. Screw back the cap.

Replacing the O-rings:

1. If the coupling should leak, the O-rings should be replaced.
2. Press the defective O-rings to one side between two fingers until a loop is formed and then slide the rings to the front and remove them.
3. Insert new O-rings in the recesses and spray them with STATCARE™.

⚠ Do not use Vaseline or any other grease.

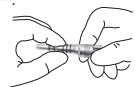CARE INSTRUCTIONS

Cleaning The Exterior:

The CONNEX™ Coupling Device can be cleaned on the outside with an alcohol solution.

⚠ Do not place the CONNEX™ Coupling Device coupling in an ultrasonic cleaner since this may result in malfunction.

DISINFECTION

The CONNEX™ coupling device can be disinfected on the outside with chemical disinfectants (spray disinfectant or wipe clean disinfectants).

The CONNEX™ coupling should not be immersed in disinfectant.

The CONNEX™ coupling must not be sterilized.

ACCESSORIES

The following are supplied by an authorized SciCan Distributor:

Description	Item #
O-rings	S150002
Replacement Bulb(s)	S150102
Tightening Tool for CONNEX L +GR	S150202

Further advice for use and safety

In case of problems, work must stop immediately.

To avoid further damage, please contact SciCan Customer Service or a SciCan authorized Dealer.

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CONNEX™, STATIS™, and STATCARE™ are trademarks of SciCan Ltd.
SciCan Inc.